



Sani-Cloth AF Universal - Information for Use (IFU)

Sani-Cloth AF Universal wipes are pre-dosed disposable disinfection and cleaning wipes for non-invasive medical devices; non-porous hard surfaces.

Uses: Disinfection for general surfaces in the Medical Area.

Directions: Dispense wipe and open, clean surface in S-motion from top to bottom of area. Ensure the surface remains visibly wet for the required contact time, use another wipe if needed. Allow surface to air dry.

Caution: Avoid contact with eyes. If irritation occurs discontinue use, if irritation persists seek medical attention. Do not use on invasive medical devices. Follow the medical device manufacturers cleaning and disinfecting guidelines. Routinely remove residual disinfectant build up. For Professional Use Only.

PPE: Use gloves for repeated or prolonged exposure.

Address: PDI Ltd, Aber Park, Flint, Flintshire, CH6 5EX

Storage: Keep product below 23°C. Store upright. Keep out of direct sunlight.


Product Usage: This product is single use. Once a wipe is dispensed from the packaging, it can only be used once and then must be disposed. Overuse and re-use will not provide the efficacy required for correct product use. Do not use if the packaging appears damaged. Do not use any wipes that are dry.

Composition: Didecyldimethylammonium chloride 0.450%w/w


Regulatory Classification: CE marked 93/42/EEC

	Product Code	Pack Format	Wipes per Pack	Packs per Case	Wipe Size	NHS Code
	XP00390	Flow wrap	200 wipes	6	267x210mm	VJT592
	XP00391	Flow wrap	40 wipes	24	200x200mm	VJT640
	XP00363	Flow wrap	100 wipes	12	280x200mm	VJT521
	XP00366	Canister	200 wipes	6	200x200mm	VJT514
	XP00365	Bucket	225 wipes	1	267x270mm	VJT513


How to Use the Wipe

1 Before you start always wear the correct PPE. 

1. Check with your internal protocols to ensure that the correct PPE is used for the product, area and equipment you are about to clean and/or disinfect.

2 Refer to label instructions and 'Use by' date. 

2. Check the expiry date.

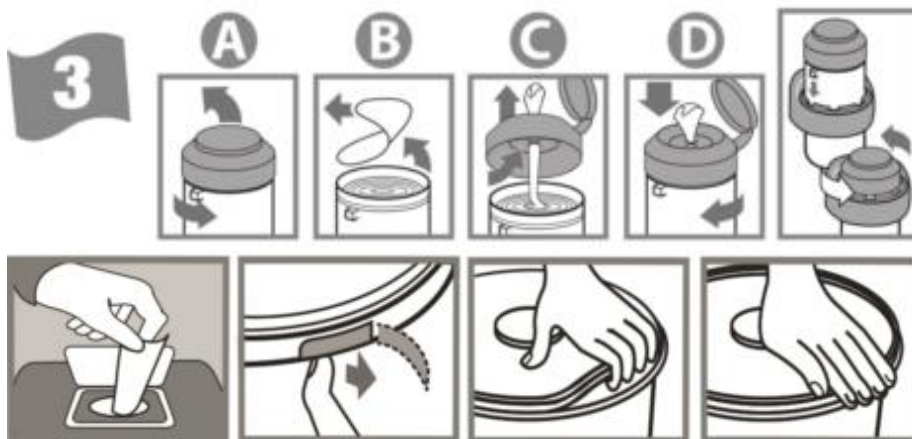

3. Instructions are available for opening the canisters, flow wrap and bucket.

4 **VISIBLY CLEAN ? VISIBLY SOILED**
Remove any visible soiling with first wipe before using additional wipes.

4. Wipe use instructions - Remove any visible soiling with the first wipe, then use an additional wipe to disinfect.



5. Remove only one wipe at a time. Open out the wipe.



6. Wipe direction should be 'dirty to clean', top to bottom, taking care not to go over the same area twice to prevent any cross contamination.



7. Contact Time & Drying Time












- One wipe covers an approximate surface area of one (1) metre square. Do not overuse the wipe, if it becomes dry or soiled discard and use another wipe to complete the area.
- Allow the disinfected area to air dry.
- Dispose of used wipes in the clinical waste bin.
- After use ensure the packaging lid is closed. Once empty, dispose of the packaging in the recycling bin, or according to local protocol


PRODUCT EFFICACY	
EN 16615 4-Field Test (Mechanical Action)	BACTERIA
Enterococcus hirae	60 sec (c)
Pseudomonas aeruginosa	60 sec (c)
Staphylococcus aureus	60 sec (c)
EN 13727	BACTERIA
Pseudomonas aeruginosa	30 sec (c)
Staphylococcus aureus	30 sec (c)
Enterococcus hirae	30 sec (c)
EN 16615 4-Field Test (Mechanical Action)	YEAST
Candida albicans	60 sec (c)
EN 13624	YEAST
Candida albicans	30sec (c)
Candida auris	30sec (c)
EN 14476	ENVELOPED VIRUS
Vacciniavirus	30 sec (c)
(HBV, HCV, HIV, H5N1, SARS, Corona)	30 sec (c)
EN 14476	NON ENVELOPED VIRUS
Norovirus	30 sec (c)
EN 14348	TB
Mycobacterium terrae	2min (c)

Product Efficacy /Product Claims: the time listed can be followed by letters that indicate the test conditions used during the testing of the product (c) for clean conditions or (d) for dirty conditions.

Packaging Symbols

Here are some of the most common symbols you may find on PDI product packaging and their description:

Title of Symbol	Description of Symbol	Symbol (Colours may vary)
Do not flush	Do not flush	
Do not macerate	Do not macerate	
Consult instructions for use	Indicates the need for the user to consult the Instructions for use, available on the website at www.pdihc.com/global	
CE mark	CE mark Medical Device Directive Class IIa	
Do not re-use	Indicates a medical device (wipe) that is intended for one use, or for use on a single patient during a single procedure	
Manufacturer	Indicates the medical device manufacturer, as defined in EU directives 90/385/EEC, 93/42/EEC and 98/79/EC	
Expressed liquid	Stability and testing completed on liquid expressed from the wipe, rather than bulk liquid.	
PAO	Period after opening – the period of time the product can be used once the product has been opened.	
Wipe count	How many wipes are in the pack.	
LOT	Allocated LOT or batch number of the product	
Expiry	The expiry date of the product	

High Density Polyethylene (HDPE)	HDPE 2 is a high density-to-strength ratio and is commonly recycled, and has the number “2” as its resin identification code. (Canisters and buckets)	
----------------------------------	--	---

Label Elements

Hazard Statements	
H412	Harmful to aquatic life with long lasting effects
Precautionary Statements	
P273	Avoid release to the environment
P501	Dispose of contents/container in accordance with local/regional/national regulation.

Authorised Representative in the European Community



NEX MEDICAL ANTISEPTICS S.r.l.
Via Per Arluno 37/39 20010 CASOREZZO (MI) Italy

FAQ's

What is contact time and what happens if the surface dries before the stated contact time on a Sani-Cloth product label?

The contact time listed on the **Sani-Cloth** product label is the total amount of time that it takes to inactivate the microorganisms listed on the product label. This time is typically referred to in minutes, and should be communicated to staff members that are utilising the disinfectant. In certain geographies and also in settings where temperature, relative humidity, and air changes may vary, it is possible that the surface may not remain visibly wet for the designated contact time. Current industry guidance requires that the treated environmental surface or equipment remains wet for the contact time stated on product label. Additional wipes may be needed in order to comply with the industry guidance, however the overall contact time does not change.

While the current industry guidance requires the treated environmental surfaces to remain wet for the stated contact time, leading researchers in infection prevention offer an alternate view. In a commentary published in Infection Control and Hospital Epidemiology (March 2018, vol. 39, no. 3, pp 229-331), Dr. W.A. Rutala and Dr. D. J. Weber suggest that contact time and treatment time are mutually exclusive. They suggest that treatment time, irrelevant of wet time, should be followed by healthcare workers for wipes and sprays (except bleach products.) PDI will continue to monitor the science closely and provide their customers with the latest information.

Following cleaning and disinfection the surface should be allowed to fully air dry.

Additional Information

Printed copies of this IFU are available upon request. Email: sales@pdi-hc.co.uk
Previous revisions of IFU's are available upon request. Email: sales@pdi-hc.co.uk
Document language: English (EN)